



# Pump Kits Cleaning and Storage

---

## BEFORE EACH USE

- **Wash hands** well with soap and warm water for 20 seconds
- **Put on gloves** and clean hospital-grade pump dials, power switch, and countertop with disinfectant wipes
- **Remove gloves and wash hands** well again
- **Assemble** clean pump kit

## AFTER EACH USE

- **Do not place your pump kit in the sink** or allow it to touch the sink
- **Clean kit parts** with dish soap and warm water over the sink
- **Rinse well** with clear water and dry with a clean paper towel
- **Put on gloves** and clean the outside of the pump and all surfaces with disinfectant wipes

## ACCESSORY KIT STORAGE

- **If unable to take your kit** with you, store it in a clean, clear container labeled with your name, department, and phone number
- **Johns Hopkins is not responsible** for items left in this room
- **Reminder:** Use only kits or parts that are your own